

Optecure[®]+CCC

Handling Instructions



HANDLING INSTRUCTIONS

1 Everything required to prepare Optecure+ccc is delivered in a single, room-temperature package:

- 1 jar of lyophilized Optecure+ccc mixture (DBM, ccc and hydrogel)
- Sterilized buffered saline solution
- 1 sterilized mixing spatula



2 Open the box of Optecure+ccc and remove the two pouches.

POUCH 1:

- The sterile inner pouch contains buffered saline solution and a mixing spatula.
- Open the outer pouch and introduce the inner pouch into the sterile field.

POUCH 2:

- The sterile inner foil pouch contains aseptically processed Optecure DBM, irradiated ccc and carrier in a plastic jar.
- Open the outer pouch and introduce the inner foil pouch into the sterile field.



3 Mixing with ABG:

- Combine Optecure+ccc with 2:1 autogenous bone. Add the measured bone into the jar of Optecure+ccc.
- Mix the contents until a putty is formed. If needed, use additional blood or buffered saline to achieve the consistency desired for each surgical situation.



4 Mixing with buffered saline:

Combine Optecure+ccc with the provided saline, stirring until completely hydrated, in putty form. This step should take about one minute.

For more robust handling, blood may be used in place of the buffered saline.



5 Remove the putty from the mixing jar, mold into desired shape and apply directly to the surgical site.



FOR REFERENCE

OPTECURE+ccc	ABG	WHOLE BLOOD OR PHOSPHATE BUFFERED SALINE
1cc	2:1	0.68cc
5cc	2:1	3.4cc
10cc	2:1	6.8cc

HANDLING INSTRUCTIONS (*cont.*)

Optecure+ccc may be hydrated with different diluents and at different volumes to achieve a variety of handling properties.

Optecure+ccc remains formable, yet resistant to lavage.

Optecure+ccc is delivered dry in a mixing jar for quick and convenient hydration. This jar may be used to combine the Optecure+ccc with required ingredients.

352-377-1140
1-800-EXACTECH
www.exac.com



713-05-01 Rev. A
Optecure+ccc Handling Instructions 0209



A Great Day in the O.R.SM