

Exactech Graft Delivery System is manufactured by Nordson Medical.
Exactech is proud to have offices and distributors around the globe. For more information about
Exactech products available in your country, please visit www.exac.com.
Rx Only

GLOBAL HEADQUARTERS
2320 NW 66TH COURT
GAINESVILLE, FL 32653 USA
+1 352.377.1140
+1 800.EXACTECH
+1 352.378.2617
www.exac.com



EXACTECH | BIOLOGICS

Handling Technique



Exactech Graft Delivery System

HANDLING TECHNIQUE

Step 1: Thread Syringe Body **A** (capacity: 15±0.5mL) onto the Tapered End Cap (white). Thread Cap Plug (blue) onto Tapered End Cap **H** (Figure 1).

Step 2: Place Funnel into top of Syringe Body. Place the Tapered End Cap (white) into the retention feature of the tray. Fill Syringe Body **A** with hydrated graft material. Remove the Funnel (Figure 2).

Step 3: Push the end of the Distal Stylet **C** into the Plunger Body **B** into Syringe Body **A**. Ensure that Plunger Body has passed the internal retention feature. Rotate to horizontal, then remove Cap Plug **G** (Figure 3).

Step 4: Thread Cannula **K** on Tapered End Cap **H** (Figure 4).

Step 5: Depress Plunger Body to dispense material to surgical site. Thread Proximal **J** stylet onto Distal Stylet **C** (Figure 5).

Step 6: Depress (push) Stylet **C** through the Cannula **K** to deliver material in the Cannula **K** to the surgical site. Use Secondary Thumb Tab **I** for longer Cannula **K** options (Figure 6).

Dispose of biohazard waste. Device will degrade if reprocessed. No effective cleaning process has been developed to prevent cross contamination. Contamination of a reprocessed device may lead to injury, illness, or death of a patient.

Figure 1



Figure 2



Figure 3



Figure 4



Figure 5



Figure 6



CLOSED GRAFT DELIVERY DEVICE PARTS LIST

- A** Syringe Body
- B** Plunger Body
- C** Distal Stylet
(contained in Plunger Body)
- D** Closed Stylet Stopper
(contained in Plunger Body)
- E** Plunger Stopper
- F** Funnel
- G** Cap Plug (blue)
- H** Tapered End Cap
- I** Secondary Thumb Tab
(10cm or 15cm Lengths only)
- J** Proximal Stylet
(5cm, 10cm or 15cm Lengths)
- K** Cannula
(5cm, 10cm or 15cm Lengths)
- L** Stylet

