

Our Science. Your Choice.

Resorbable type I collagen matrix
that provides a non-constricting
encasement for injured tendons.

TenoMend™


Collagen Tendon Wrap



TenoMend™

Collagen Tendon Wrap

TenoMend™ Collagen Tendon Wrap is a resorbable type I collagen matrix that provides a non-constricting encasement for injured tendons. It is designed to be an interface between the tendon and the tendon sheath or the surrounding tissue. When hydrated, TenoMend is a conformable, nonfriable, self-curling collagen sheet designed for easy placement under, around or over the injured tendon. TenoMend provides a protected environment and gliding surface of the sheath for tendon healing.

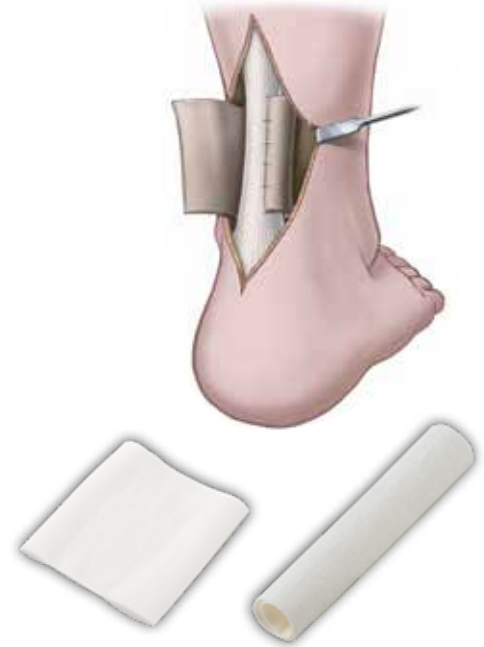

Features

- Patented self-curling design
- Proper alignment during surgery
- Convenient storage at room temperature
- Provides protective environment and gliding surface

Catalog No.	Wrap Size	Length	Diameter of Injured Tendon*
CTW16050	16.0mm	5.0cm	6.0-12.0mm, 16.0mm max*

*25% overlap is recommended. The max diameters require the wrap to meet end-to-end, which may require a running suture technique.

A rabbit flexor tendon repair study was conducted at the University of Mississippi Medical Center, Jackson, Mississippi. The study investigator was Feng Zhang, MD, Ph.D., Professor in the Department of Surgery. The results of the range of motion assessment are shown in the following table where the measurements are the joint flexion of digits (degree, mean \pm SD).¹

Range of Motion	Control Group (n=6)	Collagen Group (n=7)	Normal Group (n=7)
PIP joint (Proximal Interphalangeal Joint)	6.5° \pm 1.64°	12° \pm 2.52°	11.17° \pm 4.40°
MP joint (Metacarpophalangeal Joint)	11.0° \pm 1.94°	18.17° \pm 4.96°	23.5° \pm 11.13°

REFERENCES: 1. Data on File at Collagen Matrix, Inc.

Collagen Tendon Wrap is indicated for the management of tendon injuries in which there has been no substantial loss of tendon tissue.

Exactech is proud to have offices and distributors around the globe.

For more information about Exactech products available in your country, please visit www.exac.com.

TenoMend™ is a trademark of and manufactured by Collagen Matrix, Inc., and distributed by